**HYDROSPEED** | **HD**  
BITE REGISTRATION

 **ITENA**<sup>®</sup>

**DESCRIPTION :**

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**DESCRIPTION :**

Hydrospeed HD BR is a high precision Polyvinylsiloxane addition-type silicone material allowing accurate bite registration for easier model construction.

**TECHNICAL DATA**

- Colour: Pink
- Packaging: Set of two 1:1 ratio cartridges, 50ml each
- Accessories: 6 green EcoMix BR mixing tips + 6 clear intra-oral tips
- Linear dimensional change < 0.2%
- Hardness (Shore-A): 90A
- Working time: 45 sec
- Time in the mouth (maximum for one complete arch): 55 sec


**INDICATED FOR USE**

- Intra-oral bite registration
- Moulding of crowns, bridges, inlays and onlays
- Bite registration for orthodontic needs


**MODE OF APPLICATION**

Hydrospeed HD BR cartridges are compatible with all dispensing gun (1:1 ratio) on the market, including the ITENA dispensing gun.


All silicone mixing tips suitable for high viscosities (6.5 mm diameter) for bite registration can be used with Hydrospeed HD BR cartridges.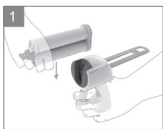


1 Insert the cartridge in the blue guide ring.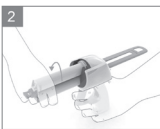

2 Lock in place with an anticlockwise twist.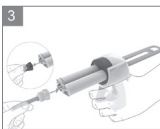

3 Remove the sealing cap and extrude a little product until the two openings are producing material at the same rate.

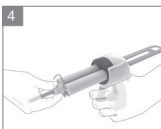4 Fit an intra-oral tip (if necessary) onto the green EcoMix BR tip and fix the assembly to the cartridge with a 90-degree clockwise twist.

5 As the lever is squeezed the material is extruded and automatically mixed in the right proportion; it may be applied directly in the mouth. Maintain steady, uninterrupted pressure. A layer of approximately 5 mm is sufficient. Flow ceases as soon as the pressure is relaxed.

6 Close the patient's jaw and keep there for the time indicated.

7 When the material is polymerised, open the patient's mouth and gently remove it. Remove any excess material from undercut areas.

8 Disinfect the dispensing gun, cartridge and mixing tip before putting away.

Do not remove the tip EcoMix BR green from the cartridge after use; leave it in place until next time, when it will be replaced. When changing the green EcoMix BR tip, check that the cartridge openings are not blocked so that the material will be able to flow freely.

**CLEANING AND DISASSEMBLY**

The dispensing gun and Hydrospeed HD BR cartridges can be cleaned using a disinfectant solution or wipe.

1 Turn the blue guide ring all the way.

2 Keep the dispenser clamp uppermost and remove the piston.

3 The dispenser disassembled.

**STERILISATION**

The Dispenser handle and piston can be autoclaved up to a temperature of 135°C/275°F.

**DISINFECTING THE IMPRESSION**

The impression may be cleaned using a disinfecting solution. Follow the manufacturer's instructions, particularly as regards time soaking in the solution so as not to risk degrading the impression. Then rinse the impression under running water to remove any trace of disinfecting solution.

**PRECAUTIONS FOR USE**

Never mix polyvinyl siloxane-based materials with silicones by condensation or with polyether impression materials. Any such combination will delay or prevent setting.

Use vinyl or nitrile gloves in preference to powdered latex, which hinders the setting of silicone-based impression materials.

Do not allow impression materials to come into contact with solvents.

**CONTRAINDICATIONS**

Use of this product can cause an allergic reaction in rare cases. Discontinue the procedure immediately and consult a doctor.

**CONSERVATION AND STORAGE**

Hydrospeed HD BR cartridges must not be used beyond their stated expiry date. They must be kept in a dry place and store at -5°C – +30°C.

For dental use only. Keep out of the reach of children

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