


# DENTO-VIRACTIS INSTRUMENTS 55

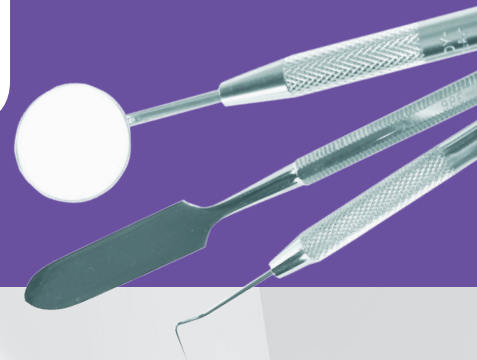

Enzymatic powder for the cleaning and pre-disinfection of highly soiled surgical instruments and medical devices prior to sterilisation.



Dilute at 0.5% : 5g per litre of tepid water (ex: For 3 litres of water, pour 15g of DV55 using the dosing device)

STEP 1



Completely immerse the instruments for 15 minutes for full microbiological efficacy

STEP 2



If needed, brush in order to eliminate the residue

STEP 3



Rinse carefully with running water

STEP 4



Dry with a clean disposable towel

STEP 6



Pack the instruments in a sterilisation pouch, then sterilise

STEP 5



## ADVANTAGES

- Enzymatic powder with high detergent power
- Enhanced cleaning properties due to the alkaline pH
- Completely water soluble
- **Cost-saving:** concentrated at 0.5% (2kg = 400L solution)
- Can also be used for impressions
- Eliminates lime scale

## CAUTION

- **Renew the cleaning bath every day.**
- Avoid the contact with aluminium materials.
- **Precautions for use:** refer to the medical safety data sheet and the label.

Product classified according to the European Directive "dangerous preparations" 1999/45/EC.

## MICROBIOLOGICAL PROPERTIES

- **Bactericidal efficacy:** EN 1040, EN 13727, EN 14561, NFT 72-301
- **Fungicidal efficacy:** NF EN 1275, EN 13624, NFT 72-301
- **Virucidal efficacy:** HIV-1, Influenza virus A/H1N1 type, Herpes virus simplex type 1 according to EN 14476