210 mm

99 mm

99 mm

99 mm

DENTOCEM

RESIN CEMENT

ITENA®

DentoCem Resin Cement

FR

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DentoCem Resin Cement

EN

Dual cured permanent adhesive resin cement. Automix.

INDICATIONS

- Final cementation of crowns and bridges, inlays and onlays, posts and cores, ceramic crowns and Maryland bridges
- Adhesive amalgam restoration
- Veneering of alloys

PROPERTIES

- DentoCem Resin Cement is a multi-purpose, dual cured, radiopaque, permanent cement which consist of two easy to mix paste components.
- It forms an excellent bond with base alloys, porcelain, dentin, enamel and with amalgams.
- The "Automix" system provides a consistent and homogeneous mixture, allows a saving in time and facilitates the use of the product.
- DentoCem Resin Cement is a self-cured cement which you can light cure for an immediate set and extra strength at the margins.
- Light curing DentoCem Resin Cement gives your restoration immediate stability, plus easy clean-up of excess material. With a simple 10 seconds light precure, you can remove excess cement before the final set. DentoCem Resin Cement features superior retention and total margin integrity.

INSTRUCTIONS FOR USE

Depending on the clinical requirement use the corresponding illustrated card.

Setting time:

- Working time at room temperature is at least 2 minutes.
- Final setting time at room temperature is approximately 5 minutes.
- Oral setting time is approximately 3.5 minutes.
- Light cure for 20 seconds.

CAUTION!

DentoCem Resin Cement base and catalyst contain polymerizable monomers which may cause allergic contact dermatitis in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs or if known allergy to methacrylate resin exists, discontinue use. Single use tips.

COMPOSITION BASE & CATALYST

- Multifunctional monomers • Photoinitiators • Glass filler • Urethan dimethacrylate

STORAGE

- For optimum freshness, keep refrigerated. Do not store above 24°C/75°F. • Keep away from direct sunlight. • Shelf life is 2 years after manufacture date.

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