



DENTO-VIRACTIS INSTRUGERM 50

Dilution:
0,5%



DENTO-VIRACTIS

Description	Dento-Viractis Instrugerm 50 is recommended for the cleaning and pre-disinfection of dental instruments. The instruments are immersed directly into the solution after use and before sterilization or chemical disinfection.
Scope of application	For use on instruments and reusable medical devices, heat-resistant and thermo-sensitive.
Characteristics	<ul style="list-style-type: none"> ✓ Dropout protein stains reinforced by the presence of surfactants with detergents and wetting strong powers ✓ Reduces the risk of cross-contamination: the dipping bath is rejected uncontaminated (7 log reduction of bacteria) ✓ Compatible with ultrasonic baths
Commercial presentation	<p>1 L bottle with 20 ml dosing device 5 L bottle</p> <p>Accessories: Pump dispenser of 10mL for 5 L bottle → for 2 L bath Pump dispenser of 15mL for 5 L bottle → for 3 L bath Pump dispenser of 20mL for 5 L bottle → for 4 L bath</p>

		Antimicrobial activity	Concentration	Contact time
Bactericidal	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Acinetobacter baumannii	0,1%	15 min
Yeasticidal	EN 13624	Candida albicans	0,5%	15 min
Virucidal	According to EN 14476	Norovirus murin	0,5%	15 min
		BVDV (surrogate of HCV) Herpes simplex virus type 1 PRV (surrogate of HBV)	0,4%	1 min
		HIV	0,4%	15 min



DENTO-VIRACTIS INSTRUGERM 50

DENTO-VIRACTIS

Composition	Cationic and non-ionic surfactants, biguanide, sequestrant and perfume.
Precautions	Contains ALCOOL ETHOXYLE C9 C11 8 OE and CHLORHYDRATE BIGUANIDE POLYMERIQUE
Instructions for use	<p>Preparation (0.5% strength solution):</p> <ol style="list-style-type: none">1. Wear protective eyewear and gloves.2. Use a disinfection tray with graded measurements and a lid.3. Dilute 5ml per litre of water (i.e. 20ml in 4l of water). <p>Usage:</p> <ol style="list-style-type: none">1. Fully immerse the (opened or dismantled) dental instruments.2. Immersion time: 15 minutes.3. Rinse thoroughly. <p>The solution must be changed as soon as it becomes dirty or at least once daily.</p>

Risk phrases:

- R38: Irritating to skin.
- R41: Risk of serious damage to eyes.
- R43: May cause sensitisation by skin contact.
- R50: Very toxic to aquatic organisms.

Security phrases:

- S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- S36/37/39: Wear suitable protective clothing, suitable gloves, and eye/face protection.
- S45: In case of accident or if you feel unwell seek medical advice immediately (show the label where possible).
- S57: Use appropriate containment to avoid environmental contamination.
- S60: This material and its container must be disposed of as hazardous waste.
- S61: Avoid release to the environment.

Refer to special instructions/safety data sheet.

Labelling:

CE Marking obtained in 2011. Version of the sheet: 3.0 of 04/10/2017