




DENTO-VIRACTIS 75 MEDICAL DEVICES

DENTO-VIRACTIS

Ready to use !

Description	Dento-Viractis 75 without alcohol is recommended for cleaning and disinfection by wet wiping .	
Scope of application	For use on non-disposable medical devices, which are not sterilizable and non-immersible.	
Characteristics	<ul style="list-style-type: none"> ✓ Disinfectant Cleaner with a broad spectrum of activity ✓ Easy to use by wet wiping ✓ Without flavor 	
Commercial presentation	750 ml spray 5 L canister	

		Antimicrobial activity	Contact time
Bactericidal	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 14561	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
	NF T 72-281	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 1276	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
	EN 13697	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Escherichia coli	5 min
Yeasticidal	EN 13624	Candida albicans	5 min
	EN 14562	Candida albicans	5 min
	NF T 72-281	Candida albicans, Aspergillus fumigatus	5 min
	EN 1650	Candida albicans	5 min
	EN 13697	Candida albicans	5 min
Virucidal	EN 14476	Rotavirus	5 min
		Norovirus	5 min
		Adenovirus	5 min
		Hepatitis B	5 min
		Hepatitis C	5 min
		VIH	5 min
Sporicidal	NF T 72-281	Clostridium difficile, Bacillus subtilis	5 min



DENTO-VIRACTIS MEDICAL DEVICES 75

DENTO-VIRACTIS

Composition	DIDECYL DIMETHYLAMMONIUM CHLORIDE. ALKYL BENZYL DIMETHYLAMMONIUM CHLORIDE
Mode d'emploi	<ol style="list-style-type: none">1. Refill the 750 ml bottle.2. Apply the product on a clean non-woven cloth or on the surface.3. Clean thoroughly the surface to be treated.4. Leave the product to act 5 minutes. Do not rinse.5. Use a fresh non-woven cloth for each equipment.

CE Marking obtained in 2015. Version 2.0 of 04/10/2017

CE HYGITECH
0 1 2 0
www.hygitech.com
109 bis avenue Charles de gaulle
92200 Neuilly sur Seine - FRANCE
Tel: +33 (0)1 48 01 32 89