100 mm

100 mm

ITENA®

BOND ACTIVATOR**Bond Activator**

FR

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Chemical activator for QuickBond.

INDICATIONS:

Bond Activator is used for chemical curing-activation of QuickBond in:

- Adhesive cementation of crown & bridges
- Adhesive cementation of inlays and onlays
- Amalgams restorations bonding
- Restorations when light-curing is not possible (in root canals)

INSTRUCTIONS FOR USE:

- 1) Dispense equal number of drops of QuickBond "B" Bond and Bond activator in a clean mixing well.
- 2) Mix for 2 seconds with a clean and new microbrush.
- 3) Apply an ample amount on the treated surface and leave undisturbed for 15 seconds.
- 4) Air-blast for at least 15 seconds or until no visible movement of the material is observed. If the surface appears incompletely coated, apply additional material and dry again.
- 5) Complete the restoration or cementation

MAIN COMPOSITION:

Ethanol • Acetone • Multifunctional monomers • Benzoyl Peroxide

CAUTION:

Bond Activator contains acetone and polymerizable monomers which may cause allergic contact dermatitis in susceptible individuals. In case of contact, wash thoroughly with water and soap. If skin sensitization occurs, or if allergy to methacrylate is known, discontinue use

STORAGE:

Do not store above 24°C (75°F)
 Keep away from direct sunlight
 Keep refrigerated for optimal conservation.
 Do not freeze
 Tightly recap after use
 Do not use after expiration date
 SDS available on www.itena-clinical.com

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