# DENTOETCH

ETCHING GEL

FR

## DentoEtch Etching Gel

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## DentoEtch Etching Gel

Etching gel - Phosphoric acid 37%

DentoEtch contains phosphoric acid for etching tooth enamel and dentin when preparing surfaces for the application of composites.

- PROPERTIES** : • High quality, non-drying, medium viscosity gel. • Stays put, does not run. • Comfortable-to-use syringe. • Tip can be angled to facilitate easy accurate placement of the gel even in distal & lingual composite preparations. • Washes off quickly & easily. • Has excellent water solubility. • Well visible blue color. • Disposable tips save time and are more sanitary.
- DIRECTIONS FOR USE** : • The effective recommended etching time is : Enamel : 30 to 60 seconds. Dentin : 15 seconds. • The DentoEtch should then be rinsed off with copious quantities of water for 30 seconds. • The disposable tip should be removed and discarded after use. • Recap syringe when not in use.
- DRYING** : • Enamel : dry adequately until the surface has a frosty white appearance. • Dentin : Dry moderately.
- CAUTION** : • DentoEtch contains phosphoric acid. Avoid contact with skin and eyes. If accidental contact occurs, flush with copious amounts of water. If swallowed, drink a large amount of water/milk and consult a doctor, do not induce vomiting.
- ESSENTIAL INGREDIENTS** : • Phosphoric acid 37%. • Water and filler 63%.
- STORAGE** : Do not store above 24°C.

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