




DENTO-VIRACTIS 77 MEDICAL DEVICES

DENTO-VIRACTIS

Description	<p>Dento-Viractis 77 is recommended For cleaning and disinfection By wet wiping.</p> 
Scope of application	<p>Use by wet wiping on surfaces (benches, trolley, furniture), equipment (monitors), non-submersible medical devices, non-sterilizable by conventional methods, invasive and non-invasive (dental units, ...). Dento-Viractis 77 is also intended for cleaning and disinfecting surfaces in contact or not with foodstuffs.</p>
Characteristics	<p>Physical state: Liquid PH: 7.0 +/- 0.5 Fragrance: grapefruit Color: colorless</p>
Commercial presentation	<p>1-liter bottle with sprayer The bottle of 5litres The box of 120 wipes The refill of 120 wipes</p>

	Essay	Target micro-organism	Contact time
Bacteria	EN 13727	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	1 min
	EN 1276	Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 13697	Pseudomonas aeruginosa, Escherichia coli, Staphylococcus aureus, Enterococcus hirae	5 min
	EN 14561	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, Acinetobacter baumannii	5 min
	EN 16615*	Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	5 min
Yeast / Fungus	EN 13624	Candida albicans	1 min
	EN 1650	Candida albicans, Aspergillus brasiliensis (niger)	5 min
	EN 13697	Candida albicans, Aspergillus brasiliensis (niger)	5 min
	EN 14562	Candida albicans, Aspergillus brasiliensis (niger)	5 min
	EN 16615*	Candida albicans	5 min



Virus	EN 14476	Norovirus, HIV-1	30 sec
	EN 14476	HBV, Herpes virus, Influenza virus type A/H1N1	1 min
		HBC, Rotavirus, Adenovirus	5 min
Spores	EN 13704	<i>Clostridium difficile</i>	15 min
	NF T 72-281**	<i>Bacillus subtilis</i>	5 min

* Standards made on wipes.

**Standards performed on sprayers (followed or not by mechanical action).

Biocide Composition	Liquid preparation and pre-impregnated wipe, TP2 / TP4, ready to use. ETHANOL CAS RN: 64-17-5 (620g / kg), N, N (3-AMINOPROPYL) DODECYLAMINE CAS RN: 2372-82-9 (1.5g / kg).
Instructions for use	<p>Bottle:</p> <ol style="list-style-type: none"> 1. Pre-impregnated a clean nonwoven with the solution. 2. Thoroughly clean the surface 3. Allow the product to work for 5 minutes (15min for C. Difficult, do not rinse (except for surfaces with food contact). 4. Change non-woven backing for each piece of equipment. <p>Wipes:</p> <ol style="list-style-type: none"> 1. Take a wipe out of its box. 2. Thoroughly clean the surface. 3. Allow the product to work for 5 minutes (15min for C. Difficult, do not rinse (except for surfaces with food contact). 4. Change the wipes for each equipment. 5. Close the box well after each use.

Version of the sheet 6.0 03/07/2019

Use biocidal products with caution

DM class IIb

Before use, read the label and product information.

Professional use