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4 seringues de 2,3g (2ml).

GB Instructions for use

PURE CARE is a water-based gel for remineralization and desensitization after a bleaching process. Pleasant peppermint flavor. Main ingredients: water, xylitol, potassium nitrate and sodium fluoride.

► INDICATIONS

Professional application: immediately after a bleaching process.
Patient application (if any): in case of ambulatory treatment.

► I) CUSTOM TRAY APPLICATION

1. Following bleach application, rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of PURE CARE into the tray and apply to the upper and / or lower teeth.
3. Keep the tray in mouth for a minimum of 3 minutes.
4. Remove the tray.

In case of ambulatory treatment:

5. Ask the patient to rinse any remaining gel on the surface. Advise the patient not to eat nor drink for 30 minutes following the application.
6. Any remaining gel in the tray should be immediately rinsed or brushed off under running water.

► II) TOOTH APPLICATION

1. If necessary, remove any saliva excess on the tooth surface with a cotton roll or a specific device.
2. Apply a generous layer of gel on the tooth surfaces using an application swab, gloved finger or interproximal tooth brushes if necessary.
3. Keep the gel in mouth for a minimum of 3 minutes.

In case of ambulatory treatment:

4. Ask the patient to rinse any remaining gel on the surface. Advise the patient not to eat nor drink for 30 minutes following the application.

► SIDE-EFFECTS

With a proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be completely ruled out. Should you learn about unwanted side effects – even if it is doubtful that the side effect has been caused by our product – please kindly contact us.

► CONTRAINDICATIONS / INTERACTIONS

If a patient has known allergies against or hypersensitivities towards a component of this product (refer to main ingredients list), we recommend not to use it or to do so only under strict medical supervision. The dentist must consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

► STORAGE

Store in a cool, dry area and away from direct sunlight.

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GDF GmbH