**PURE
PROTECT**
GINGIVA PROTECTOR
Gel de protection gingivale

aitena

**PURE
PROTECT**
GINGIVA PROTECTOR
Gingiva protection

aitena

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GB Instructions for use

PURE PROTECT is a gingiva protection used to isolate the gingiva from the teeth to be treated.

► INDICATION

Protect the gingiva during the in-office bleaching treatment.

► CONTRAINDICATION

This product contains dimethacrylates. In case of allergy or intolerance to this component, do not use PURE PROTECT.

► PREPARATION

Take appropriate measures (for example cheek and lip retractors) to ensure that the tongue and lips are kept in a safe position during the treatment (and the subsequent bleaching procedure) and is not in contact with the material. Remove the cap of the syringe and replace it by a bent application tip. Regulate the material's flow.

► APPLICATION

Dry the gingiva. Apply PURE PROTECT to a width of approx. 4-6 mm and a thickness of 2mm along the area, and also extrude the material into the interdental spaces. Light-cure during 20 to 40 sec using a standard light-curing lamp. Do not hold the lamp too long on one area. Remove any material excess on the teeth using a scalpel. The bleaching gel can now be applied.

► FINISHING

After bleaching, remove PURE PROTECT by lifting the material at one end with an instrument, such as a probe, and pull it off. Remove the syringe bent application tip and replace it by the cap.

► SHELF LIFE/STORAGE

PURE PROTECT should not be used after the expiry date. Store the product in a dark place. Avoid heat and direct sunlight.

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