70 mm

70 mm

70 mm

## QUICK BOND

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Universal, no rinse self-priming etchant adhesive system. EN

### INDICATIONS :

- Direct composite restorations
- Adhesive cementation of crowns & bridges
- Adhesive cementation of inlays and onlays

### PROPRIÉTÉS :

Quick Bond is a 6<sup>th</sup> generation self-etching bonding system, which deliver bonding properties to both dentin and enamel. Quick Bond consists of an acidic water-based self-etching primer, and a light-curing adhesive. It is used to bond resin-based filling composites, self-curing or light-curing composites, restorative compomers and resin-based composite cements to tooth structure.

### INSTRUCTION FOR USE :

Application of QuickBond "A" Prime:

- 1) Apply 3 successive layers of QuickBond "A" Prime on the prepared and air-dried tooth surface. Leave undisturbed for 15 seconds.
- 2) Air-blast for 10 seconds in order to remove solvents residues.

Application of QuickBond "B" Bond:

- 3) Apply an ample amount of QuickBond "B" Bond
- 4) Air-blast for at least 10 seconds or until no visible movement of the material is observed. If the surface appears incompletely coated, apply additional material.
- 5) Cure for 20 seconds using a light-curing unit.
- 6) If the retentive surface lies within dentin, apply an additional layer of QuickBond "B" Bond and air-blast for at least 10 seconds. Do not cure.
- 7) When light-curing is not an option (when bonding in root canal), add equals drops of Bond Activator with QuickBond "B" Bond in a clean mixing well, mix for 2 seconds and proceed with steps 3, 4 and 7.
- 8) Restore with direct resin-based composite of choice. For indirect-restorations, use QuickBond in conjunction with Dentocem resin cement.

### MAIN COMPOSITION :

Acetone • Ethanol • Water • Photoinitiators • Multifunctional monomers & methacrylates • Urethan dimethacrylate

### CAUTION :

Quick Bond contains polymerizable monomers which may cause allergic contact dermatitis in susceptible individuals. In case of contact, wash thoroughly with water and soap. If skin sensitization occurs, or if allergy to methacrylate is known, discontinue use.

### STORAGE :

Do not store above 24°C (75°F)  
Keep away from direct sunlight  
Keep refrigerated for optimal freshness.  
Do not freeze  
Tightly recap after use  
Do not use after expiration date  
SDS available on [www.itena-clinical.com](http://www.itena-clinical.com)

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